Gundersen Lutheran Laboratories Lacrosse and Regional, POCT Accu-Chek Inform Knowledge Test

Name	: Employee#:
Unit/I	Dept Date:
-	Review: I have reviewed the Point of Care Blood Glucose Testing policy and esponsibility for knowing its contents. Initial and Date of Review
The po	olicy can be accessed on the intranet by following the following steps:
1. 2. 3. 4.	Log onto the intranet, Gladiator. Select Policies and Procedures. Type the words "glucose meter" in the search line. Select Search. Click on the specific policy – Lab-0910 <i>Point of Care Blood Glucose Testing</i>
	r the supervision of the certified instructor, the operator will be able to lize/demonstrate the following:
	Turn on the meter and verify QC status and battery status-Code Key Replacement. Scan the operator ID number from name badge. Perform a low-level and a high-level quality control test- Demonstrate correct control handling steps and horizontal dosing technique. Use barcode scanner. Enter the patient ID number – Review Source of MRN/Clinic#- Drop Leading Zeros-Include Check Digit. Note: CSN# should not be used for Patient ID. Review Use of Positive Patient ID-Second Identifier. Obtain adequate blood sample following correct blood collection procedure Enter appropriate comments-Chart Result vs. Do Not Chart and Critical Notification comments. Dock the instrument after each use. Review storage of test strips and control solutions- Expiration dates and labeling. Use of memory- review results from meter's database. Review of 1234567 guidelines- Comment Choice and Required Correction Form. Meter cleaning with proper documentation. Isolation procedure - Use test strips within 3 minutes. Troubleshooting Form- Loaner units available.
	er of correct answers on the Accu-Chek Inform Knowledge test correct = 30 or 29 / <25 = Retest)
	perator has successfully participated in a training session and I have reviewed all incorrect answers e operator. They have achieved six-month certification status. 6 month recertification is due:
_	ture of the Certified Instructor:Date: w and Recorded by:Date: