Health o meter®

P r o f e s s i o n a l

500KL

PROFESSIONAL DIGITAL BEAM SCALE
BALANZA ELECTRONICA PROFESIONAL
BALANCE ELECTRONIQUE PROFESSIONNELLE

500 lb x 0.2 lb
220 kg x 0.1 kg

ISO13485:2003
CERTIFIED

Rev. 20100625

CE 0470

CSA®
C US

232791
IEC 60601-1
C22.2 No. 601.1-M90
UL 60601-1

PATENT D565446 (US)

USER INSTRUCTIONS • INSTRUCCIONES DE USO • MODE D'EMPLOI

Thank you for your purchase of this Health o meter[®] Professional scale.
Please read this manual carefully and keep it for easy reference or training.

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TABLE OF CONTENTS

ENGLISH	2
ESPAÑOL	8
FRANÇAIS	14
CAUTION AND WARNING	3
SYMBOLS AND SPECIFICATIONS FOR THIS SCALE	3
CONNECTIVITY INFORMATION	3
CERTIFICATIONS / DISPOSAL	4
SET UP / BATTERIES	4
QUICK START INSTRUCTIONS	5
OPERATION OF THIS SCALE	5
MAINTENANCE	6
CALIBRATION	6
TROUBLESHOOTING	6
WARRANTY	7

PLEASE REGISTER YOUR SCALE FOR WARRANTY COVERAGE AT:
<http://www.homscales.com/customers/registration.aspx>

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CAUTION AND WARNING

To prevent injury and damage to your scale, please follow these instructions very carefully:

- Do not transport the scale while platform is loaded. Be sure the scale is tilted enough to assure it is rolling properly on its wheels when it is moved for cleaning or transported from one area to another.
- For accurate weighing the scale must be placed on a flat, stable surface.
- For accurate weighing, verify proper operation according to the procedure described in this manual before each use.
- Do not use in the presence of flammable materials.
- Operating at other voltages and frequencies than specified could damage the equipment.
- If the “LOW BAT” indicator activates, for accurate weighing replace the batteries or connect the scale to an AC power source as soon as possible.
- Only manufacturer’s qualified personnel shall service the scale.

This equipment has been tested and found to comply with the EMC limits for the Medical Device Directive 93/42/EEC (EN 55011 Class A and EN 60601-1-2). These limits are designed to provide reasonable protection against harmful interference in a typical medical installation. The equipment generates, uses and may radiate radio frequency energy and, if not installed and used in accordance with these instructions, may cause harmful interference to other devices in the vicinity. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference with other devices, which can be determined by turning the equipment off and on, the user is encouraged to try to eliminate the interference by one or more of the following measures:

- Reorient or relocate the receiving device.
- Increase the separation between the equipment.
- Connect the equipment into an outlet on a circuit different from that to which the other device(s) is connected.
- Consult the manufacturer or field service technician for help.

DEFINITION OF SYMBOLS


Attention, Consult Accompanying Documents

Type BF Applied Parts


AC to DC power supply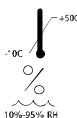

Temperature range
(50 °F to 104 °F / 10 °C to 50 °C)


Humidity range
(10% - 95% RH)

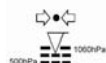Atmospheric pressure range
(500hPa – 1060hPa)

AC to DC power supply
(9V DC, 200mA)

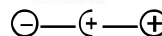DC Connector Polarity

500KL SCALE SPECIFICATIONS

Capacity and Resolution:	500 lb x 0.2lb / 220 kg x 0.1 kg
Power Requirements:	6-AA cell 1.5V batteries (INCLUDED) or optional Adapter Model RHD090020/ADPT31 (USA - CSA only) 120v AC – 9V DC 60 Hz or with Adaptor Model UE 15WCP-090050SPA/ADPT30 (IEC only) 120-240V AC – 9V DC 50-60Hz. OPTIONAL ADAPTERS NOT INCLUDED.
Environmental:	Operating Temperatures: 50 °F to 104 °F / 10 °C to 50 °C Atmospheric Pressure: 500hPa to 1060hPa Humidity: 10% to 95% RH
Platform Dimensions:	12-3/4" x 15-1/2" x 2 1/4" / 32.4 cm x 39.4 cm x 5.7 cm

STORAGE SPECIFICATIONS

Storage Temperatures:	32°F to 122°F / 0°C to 50°C
Humidity:	10% - 95% RH
Atmospheric Pressure:	500hPa – 1060hPa

*WARNING:

For use with AC Adaptor Model RHD090020/ADPT31 (USA - CSA only) or with Adaptor Model UE 15WCP-090050SPA/ADPT30 (IEC only). Complies with C22.2 No. 601.1-M90 and UL Std No. 60601-1. Type BF applied parts. Internally powered. **Equipment is not suitable for use in the presence of flammable anesthetic mixture with air nor with oxygen or nitrous oxide.**

500KL SCALE CONNECTIVITY INFORMATION:

This scale is designed to connect to other devices via a USB port, to allow acquisition of weight, height and BMI from the scale, to help close a source of potential medical errors caused by incorrectly copying and then recording a weight taken. This scale is EMR-EHR compatible. For more information regarding how to connect this scale to your computer, a monitor or other device please contact Health o meter® Professional Technical Support at (+1) 800-638-3722.

500KL CERTIFICATIONS / DISPOSAL

CERTIFICATION DESCRIPTIONS:

	Products carrying this symbol meet the requirements of: 1. Directive for non-automatic weighing instruments 90/384/EEC 2. Directive governing medical devices 93/42/EEC
	Pelstar, LLC has been officially certified as a manufacturer of Health o meter® Professional medical devices to be in compliance with stringent legal requirements. The Pelstar, LLC quality assurance system covers the development, production, sales and service of medical scales and measuring systems.

500KL SCALE DISPOSAL:

Do not dispose of this device in domestic waste. This Health o meter® Professional scale must be disposed of properly as electronic waste. Follow the national, regional or local regulations which apply to you for disposal. For scale assistance or telephone support, please call Health o meter® Professional Customer Service (available on normal business days) at (+1) 800-815-6615 or (+1) 708-377-0600.

SET UP / BATTERIES

INSTALLING BATTERIES

1. Locate the battery compartment on the back of the display head.
2. With a Phillips screwdriver remove the screw from the battery compartment cover and remove cover.
3. Place 6 new AA alkaline batteries into the battery compartment. Make sure the '+' and '-' are positioned correctly, as shown on the bottom of the battery compartment. **Replace the batteries when the display reads 'LobAt'** (Use only 6 AA alkaline batteries).
4. Replace the battery compartment cover.

NOTE: Dispose used batteries in accordance to your country's national, state or local requirements.

OPERATION WITH ADAPTER POWER

Plug the optional adapter (NOT INCLUDED) into the base of the scale and then into the outlet for power. Please refer to page 3 for information regarding the adapters available for use with this scale. When an adapter is used, the battery icon will not be displayed. The location of the adapter power jack on the scale (C) is shown below in Figure 1.

ASSEMBLING AND PREPARING THE SCALE FOR USE

1. Locate cable inside the upper column (A) and then feed it through the lower column (B).
2. After the cable is fed through (B), then plug the cable into the scale base (C).
3. Connect columns (A) and (B), then secure to base (C), using the 4 screws provided (see Figure 1).
4. Connect upper height rod (E) to lower height rod (F), then slide it into the slot at the top of the upper column (H).
5. Turn the scale on by pressing the ON/OFF button. The display will show "Start" and then "0.0".

NOTE: If set up instructions failed, refer to the troubleshooting instructions. If troubleshooting does not resolve the problem, please contact Health o meter® Professional Technical Support at (+1) 800-815-6615 or (+1) 708-377-0600.

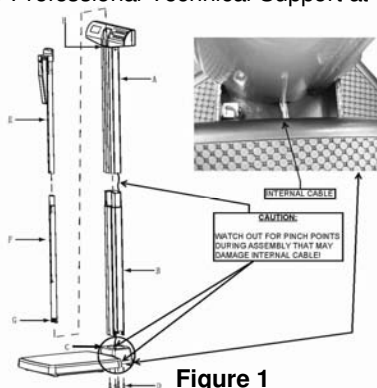
Figure 1

QUICK START INSTRUCTIONS

1. Press the ON/OFF button to turn the scale on. "START" will appear on the display for 4-5 seconds, then "0.0" will be displayed.
2. Have the patient step on the scale. After the weight has stabilized, "LOCK" and the patient's weight will be displayed.
3. The stable weight will continue to be displayed until the patient steps off the scale. The display will again show "0.0".

DESCRIPTION FUNCTION

ON/OFF	Turns the scale ON and OFF.
HOLD/RELEASE	Holds the value of the weighed object on the display until the button is pressed again.
ZERO	Zeros the display prior to weighing; enables the cancellation of weight if some object was placed on the scale. Can be used if you want to tare that object.
LB/KG	Toggles between pound or kilogram uses of the scale.
CLEAR	Clears the incorrect height input when in BMI mode.
BMI	Prompts entry of data to calculate the patients Body Mass Index (BMI)
ENTER	Used to accept height in BMI mode.
UP ▲	Increases height input (0.5 in / 0.5 cm increments)
DOWN ▼	Decreases height input (0.5 in/ 0.5cm increments)

OPERATING INSTRUCTIONS

AUTO OFF FUNCTION

As an energy saving feature, the scale will automatically turn off after 2 minutes of inactivity, if this function is enabled.

Activating auto-off

1. While the scale is turned off, press and hold the "HOLD/RELEASE" button.
2. While holding the "HOLD/RELEASE" button, press the "ON/OFF" button to turn the scale on.
3. "START" will briefly appear on the display followed by "AOF=1".
4. Auto off is now enabled.

De-activating auto-off

1. While the scale is turned off, press and hold the "HOLD/RELEASE" button.
2. While holding the "HOLD/RELEASE" button, press the "ON/OFF" button to turn the scale on.
3. "START" will briefly appear on the display followed by "AOF=0".
4. Auto off is now disabled and the scale will remain on until the scale is manually turned off.

KG & LB LOCK OUT

The default weighing mode for this scale can be changed from LB/KG to KG only or LB only.

To change weighing mode:

1. Press the ON/OFF button to turn the scale on.
2. Wait for "0.0" to appear on the display.
3. Press and hold the LB/KG button until the unit of measure begins to flash.
4. Release the LB/KG button and the scale will operate in the mode you selected.
5. Repeat steps 3-4 to change the weighing mode to a different unit of measure as the default for use.

BMI CALCULATION

1. Complete steps 1 and 2 of the Quick Start Operating Instructions, then do the following steps.
2. While the patient is standing on the scale, press the BMI button. A number representing a default height will appear on the display.
3. Select the correct or closest height measurement by pressing the UP or DOWN buttons.
4. Confirm the height input by pressing the ENTER button and the BMI will be displayed.
5. Press the CLEAR button to return to normal weighing operation. See key descriptions above for more information.

HOLD FUNCTION

1. Complete steps 1 and 2 of the Quick Start Operating Instructions, then do the following steps.
2. While the patient is standing on the scale, press the HOLD/RELEASE button to activate the hold function. The stable weight will remain on display for convenience until the hold function is cancelled.
3. To cancel hold function, press the HOLD/RELEASE button again. The scale will return to normal weighing operation.

Before first use and after periods of non-use, check the scale for proper operation.

Disconnect the scale from the power source, if plugged in.

1. This scale contains electronic parts. Use it carefully. Do not drop, kick or jump on it.
2. Do not store near chemicals, heaters or radiators.
3. Do not store scale on its side.
4. To clean the scale, wipe the outside of the scale with a soft cloth and mild detergent. Do not use abrasive cleaners. Do not use isopropyl alcohol, other solutions or sanitizing wipes to clean the display surface.
5. Never place the scale in the water. Do not spray it with liquid.

This scale is an extremely sensitive weighing device. Items such as clothing or towels placed or dropped on the scale may cause it to activate. In order to prevent run down of the battery, do not store or leave anything on the scale.

CALIBRATION

This scale has been factory calibrated and does not require calibration prior to use.

The calibration procedure may be performed using a known weight in either pounds or kilograms. Simply set the scale for the desired unit of measure using the lb/kg key and start the calibration procedure. The calibration is performed using lb or kg, according to the unit of measure used upon entering into the calibration path, meaning the scale will default to either lb or kg, if that was the setting left on the scale for weighing.

Note: Calibration weight must be no less than 100 lb (40 kg) and no more than 500 lb (200 kg).

ACTION	DISPLAY SHOWS
Press and hold the LB/KG and ZERO buttons at the same time, then press the ON/OFF button to turn on the scale. "CAL" followed by a number indicates you can calibrate.	"CAL" then "XXXXX"
Press the ZERO button. "C-100" (for lb mode) or "C-40" (for kg mode) will flash on the display and then the display will return to "0".	"C-100" or "C-40"
Place 100 lbs or 40 kg on the scale and when calibration number stabilizes, press the ZERO button. "C-200" (for lb mode) or "C-80" (for kg mode) will flash on the display and then the display will return to "0".	"C-200" or "C-80"
Place 200 lbs or 80 kg on the scale and when calibration number stabilizes, press the ZERO button. "C-300" or "C-120" will appear on the display and then the display will return to "0".	"C-300" or "C-120"
Place 300 lbs or 120 kg on the scale and when calibration number stabilizes, press the ZERO button. "C-500" or "C-200" will appear on the display and then the display will return to "0".	"C-500" or "C-200"
Place 500 lbs or 200 kg on the scale and when calibration number stabilizes, press the ZERO button. "END" will appear on the display and then the scale will automatically turn off.	"END"

TROUBLESHOOTING

Refer to the following instructions to check and correct any failure before contacting service personnel.

SYMPTOM	POSSIBLE CAUSE	CORRECTIVE ACTION
Scale does not turn on	<ol style="list-style-type: none"> 1. Battery problem 2. Bad Power Supply or Not Plugged In 3. Electrical Outlet not functioning 	<ol style="list-style-type: none"> 1. Check / Replace the batteries 2. Check / Replace Power Supply Adapter 3. Use a different outlet
"LobAt" shows in display	The batteries are low	Replace batteries according to instructions
Questionable weight or the scale does not zero	<ol style="list-style-type: none"> 1. External object interfering with the scale 2. The display did not show "0.0" before weighing 3. Scale is not placed on a level surface 4. Scale is out of calibration 	<ol style="list-style-type: none"> 1. Remove interfering object from the scale 2. Ask the patient to step off the scale. Zero the scale and begin weighing process again. 3. Place scale on level surface 4. Check weight with know weight value
"OVER" shows in display	Overload. The load on the scale exceeds scale capacity (500 lbs / 220 kg)	Remove weight, turn scale off, turn scale on again and try again
"UNDER" shows in display	Negative weight, indicating a tare was present. Load on the scale is below initial zero point	Clear scale of any weight and re-zero scale by pressing ZERO button

What does the Warranty Cover?

This Health o meter® Professional scale is warranted from date of purchase against defects of materials or in workmanship for a period of two (2) years. If product fails to function properly, return the product, freight prepaid and properly packed to Pelstar, LLC (see "To Get Warranty Service", below, for instructions). If the manufacturer determines that a defect of material or in workmanship exists, the customer's sole remedy will be repair or replacement of the scale at no charge. Replacement will be made with a new or remanufactured product or component. If the product is no longer available, replacement may be made with a similar product of equal or greater value. All parts including repaired and replaced parts are covered only for the original warranty period.

Who is Covered?

The original purchaser of the product must have proof of purchase to receive warranty service. Please save your invoice or receipt. Authorized dealers selling Pelstar products do not have the right to alter, or modify or any way change the terms and conditions of this warranty.

What is Excluded?

Your warranty does not cover normal wear of parts or damage resulting from any of the following: negligent use or misuse of the product, use on improper voltage or current, use contrary to the operating instructions, abuse including tampering, damage in transit, or unauthorized repair or alternations. Further, the warranty does not cover Acts of God, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights, and you may also have other rights that vary from country to country, state to state, province to province or jurisdiction to jurisdiction.

To get Warranty Service make sure you keep your sales receipt or document showing proof of purchase. Call (+1) 800-638-3722 to receive a return authorization (RA) number, which must be included on the return label. Attach your proof of purchase to your defective product along with your name, address, daytime telephone number and description of the problem. Carefully package the product and send with shipping and insurance prepaid to:

Pelstar, LLC
 Attention R/A# _____
 Repair Department
 11800 South Austin Avenue - B
 Alsip, IL 60803 USA

If your scale is not covered by warranty, or has been damaged, an estimate of repair costs or replacement costs will be provided to you for approval, prior to servicing or replacing.

PELSTAR, LLC

11800 South Austin Avenue – B • Alsip, IL 60803 • USA

1-800-638-3722 or 1-708-377-0600

www.homsscales.com

**PLEASE REGISTER YOUR SCALE FOR WARRANTY COVERAGE AT:
<http://www.homsscales.com/customers/registration.aspx>**

Health o meter® Professional products are manufactured, designed and owned by Pelstar, LLC. Health o meter® is a registered trademark of Sunbeam Products, Inc. used under license. We reserve the right to improve, enhance or modify Health o meter® Professional product features or specifications without notice.

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© Pelstar, LLC 2009