THUASNE

LOMBAFIX[®]

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Composition :

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REINFORCED LUMBAR SUPPORT BELT, WITH PADDING

(ENG)

Properties :

- Aerated elastic CombiteX® fabric (patented).
- Crossed-over straps of fabric. (1)
- Removable pad incorporating rigid stays fitting anatomically, and dimpled surface giving a massage effect (the pad is fixed in place by self-adhesive straps). (2)

Indications :

- Progressive support of acute lumbar pain and sciatica.
- Post-operative progressive lumbar support.

Fitting instructions :

- Using the velcro strap, position the pad vertically, well centred on the dorsal part, the wide part beside the belt's label.
- Put on the belt, with the inside label upwards.
- Hold the left end flat using your left hand.
- Fasten the velcro strap, adjusting it to the required support (if the velcro strap is properly positioned it does not wrinkle).

Precautions :

Follow the advice of the professional who prescribed or supplied the product. It may be necessary to reshape the stays. Consult your professional if discomfort occurs.

Store at room temperature, preferably in its original packaging.

Re-use of this device for the treatment of another patient is strictly not recommended due to hygiene- and performance-related problems. If a decision is taken to re-use this device, then the person taking the decision is solely responsible and the Thuasne warranty will automatically become void.

Washing instructions :

- Remove the pad and its velcro strap before washing.
- Wash only with soapy water, without detergent or bleach.
- Squeeze out excess water.
- Dry away from heat sources.

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