



**BEFORE**



**AFTER**

## HEAD AND NECK POSITIONING AND SUPPORT SYSTEM

The i2i System was developed by Leslie Fitzsimmons PT, STP who after trying all available options decided that enough was enough and designed the i2i System for her most challenging students at Lakeview School in New Jersey.

## WHY THE i2i WORKS?

The unique integrated form (shape) of the i2i connects the head and neck and forms a support similar to that of an exoskeleton.


Nor does it position low on the neck similar to many collar type devices which tend to leave the upper neck and head unsupported they are free to exhibit the impact of weakness or tone.

The anterior arms act as an anchor for the Chin Prompt, provide an anterior thoracic boundary and a channel to direct extensor tone into a midline position.

The lateral component promotes midline head position and offers cervical support while maintaining range of motion. As a client begins to extend, their head is invited back into a more neutral position. A midline position discourages reflex postures and provides appropriate proprioceptive input. It has also been found to promote the development of controlled neck flexion & extension.



**i2i with Chin Prompt**

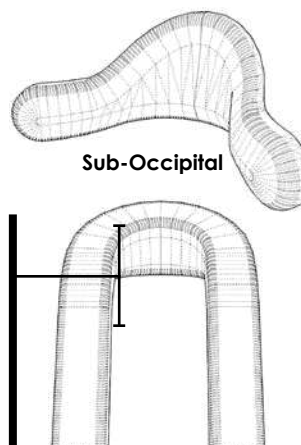

**BEFORE**




**AFTER**



## CLINICAL GUIDELINES

This clinical guideline should be used as a general reference for product applications. Final decisions regarding size, material, foam and gel placement should be left to qualified individuals who are familiar with a client's specific needs through evaluation and/or trial and observation.

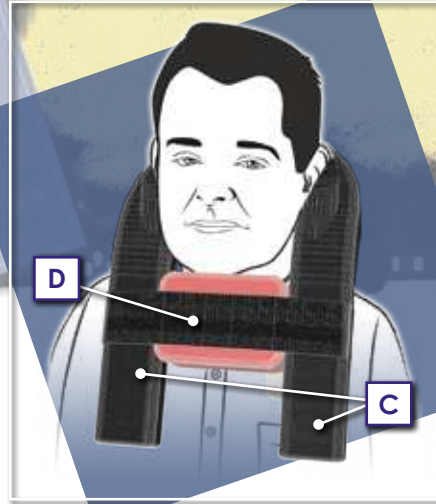

**A** The unique, integrated shape of the i2i provides a maximal, proximal foundational connection between the head and neck. The sub-occipital and integrated low profile occipital components provide posterior support and set a limit to cervical extension while also offering a resting spot.

**B** Anterior arms act as an anchor for the chin prompt and provide an anterior thoracic boundary that channels movement into a midline position. With children, it is especially important the anterior arms don't obstruct shoulder movement.

**C** The lateral component promotes midline head position and offers cervical support while maintaining range of motion (ROM). Individuals with excessive weakness and poor head control should choose a more intimate fit while individuals with more controlled movement may choose a looser fit.


**D** The chin prompt is a dynamic support that provides foundational anterior head positioning. When systematically raised and lowered, it is an excellent therapeutic tool that encourages neck muscle strengthening. The chin prompt was carefully crafted and engineered with a curved posterior that prevents airway obstruction and a width that prevents an individual's chin from rolling over the front. Additionally, the prompt is made with a closed cell foam that prevents sinking and the straps are designed to slide if too much pressure is exerted. A variation of the chin prompt is also offered that is specifically designed for traches. For safety, the chin prompt should only be used under a caregivers supervision, and is **NOT** to be used during transportation.

**E** Individuals with excessive lateral flexion should consider ear cutouts on the i2i. The cutouts are specifically designed to increase ROM while reducing excessive lateral flexion and rooting reflexes. Also, ear cutouts should lessen friction and pressure for those with severe lateral flexion thereby reducing irritation and deformity.

Swing-Away facial lateral pads can be used as needed in conjunction with the i2i for added anterior or lateral support. Switches can either be used for easy access, or positioned to promote active head and neck control.

# i2i ORDER FORM

Customer Name:			Purchase Order No.:		
Contact:			Date:		
			Mark of:		
Bill To:			Ship To:		
Address:			Address:		
City:	State:	Zip-Code:	City:	State:	Zip-Code:
Phone:			Phone:		
Fax:			Email:		

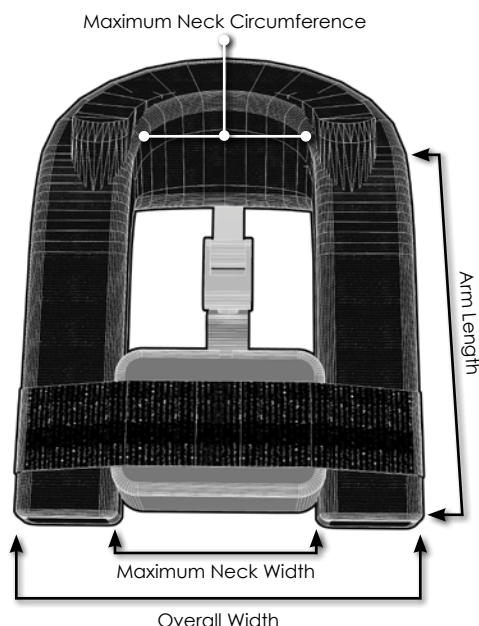

Part Number	Maximum Neck Circumference	Overall Width	Arm Length	Maximum Neck Width
i2i 480 - Small Standard	11"	8"	6.5"	4"
i2i 481 - Small Short Arm	11"	8"	4.5"	4"
i2i 580 - Medium Standard	14"	9"	7.5"	4.5"
i2i 581 - Medium Short Arm	14"	9"	5.5"	4.5"
i2i 680 - Large Standard	16"	9.5"	8"	5"
i2i 681 - Large Short Arm	16"	9.5"	6"	5"
i2i 780 - X-Large Standard	18"	10"	9"	5.75"
i2i 781 - X-Large Short Arm	18"	10"	6.25"	5.75"

**Fabric:**  
i2i Head Rest: Dartex, Reverse Dartex, Neoprene, and white or black Sheepskin.  
Chin Prompt: Neoprene or Vinyl.

Part Number	MSRP	Part Number	MSRP
<input type="checkbox"/> i2i 480 Small Standard	\$699.00	<input type="checkbox"/> i2i 582 Medium Standard w/Gel	\$868.00
<input type="checkbox"/> i2i 481 Small Short Arm	\$699.00	<input type="checkbox"/> i2i 583 Medium Short Arm w/Gel	\$868.00
<input type="checkbox"/> i2i 482 Small Standard with Gel	\$868.00	<input type="checkbox"/> i2i 680 Large Standard	\$699.00
<input type="checkbox"/> i2i 483 Small Short Arm with Gel	\$868.00	<input type="checkbox"/> i2i 681 Large Short Arm	\$699.00
<input type="checkbox"/> i2i 580 Medium Standard	\$699.00	<input type="checkbox"/> i2i 682 Large Standard with Gel	\$868.00
<input type="checkbox"/> i2i 581 Medium Short Arm	\$699.00	<input type="checkbox"/> i2i 683 Large Short Arm with Gel	\$868.00

Coding: The i2i is a Head & Neck Positioning System, designed to help position clients exhibiting neck extension/flexion (high tone) or neck weakness (low tone). CODE: K0108

**SEE REVERSE FOR CUSTOM AND OPTIONAL NEEDS**



Mail this order form to:  
**Stealth Products, Inc.**  
P.O. Box 458  
103 John Kelly Drive  
Burnet, TX 78611

Or Send it to:  
**Fax: 1.800.806.1225**

For more information call:

**1.800.965.9229**








# MEASURING CUSTOMIZATION INFORMATION

## i2i Drawing to be marked for Customization Needs.

- For example:
- If you need foam build-up or removal
  - If you want get padding on selected areas (other than shown standard)
  - If you need custom arm length

### Measure Arm Length Fit:

1. Start at point A
2. Measure to Point B

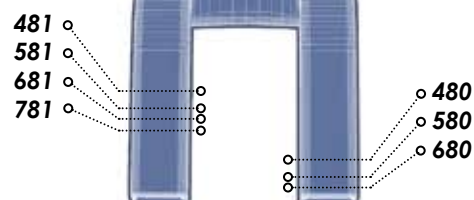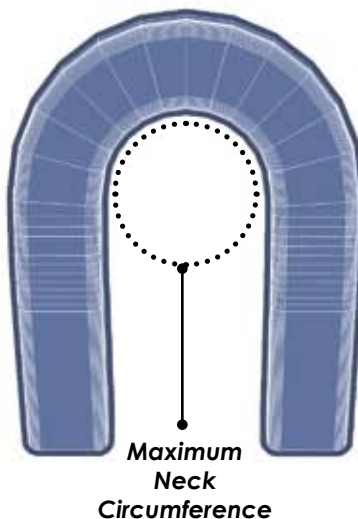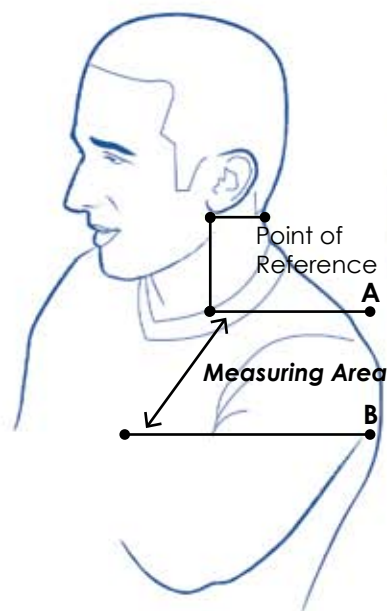

\* Part Number is for one side only. Order 2 for both side

**Mark-Up  
Drawing**

**Standard  
Gel  
Inserts**



**Stealth Products, Inc.**  
P.O. Box 458  
103 John Kelly Dr, Burnet, TX 78611  
1.800.965.9229 • Fax 1.800.806.1225  
[www.StealthProducts.com](http://www.StealthProducts.com)

(See Order Form for cost and details)